

APPROVED CLEANING PRODUCTS FOR MATTRESS COVERS, UPHOLSTERY TOPS AND SHOWER TROLLEY TUB FABRICS:

APPROVED CLEANING PRODUCTS	MAXIMUM CONCENTRATION
Actimax	Use according to the manufacturers specifications.
Arjo Disinfectant	Use according to the manufacturers specifications.
Clinell Hand & Surface Sanitizer Disinfectant	Use according to the manufacturers specifications.
Clinell Universal Sanitising Wipes	
Clinell Detergent Wipes	
Clinell Sporidical Wipes (GlobalMedics)	
Ecopine	5% Ecopine (1 part Ecopine to 20 parts water)
Leggeside HB (Legge Systems)	Use according to the manufacturers specifications.
Mediwipes Disinfection (Sulco Limited)	Use according to the manufacturers specifications.
PAL Detergent Wipes	Use according to the manufacturers specifications.
PAL Disinfectant Wipes (GlobalMedics)	
PureGreen 24 (Pure Green)	Use according to the manufacturers specifications.
Sanicare TBX	Use according to the manufacturers specifications.
Sanicare Disinfectant Wipes (Buckeye International)	
Spring Clean (Eco Lab)	5% Spring Clean (1 part Spring Clean to 20 parts water)
Super Sani Cloth -Germicidal Disposable Wipes (PDI)	Use according to the manufacturers specifications.
Surface Sanitising Wipes	Use according to the manufacturers specifications.
Neutral Detergent Wipes (Reynard Health Supplies)	
Terminator One Step Disinfectant (Buckeye International)	Use according to the manufacturers specifications.
Trigene (Bio Strategy)	Use according to the manufacturers specifications.
Tuffie 5 Cleaning Wipes (Vernacare)	Use according to the manufacturers specifications.
Viraclean (Whiteley Medical)	Use according to the manufacturers specifications.
Virkon (Dupont)	5% Virkon (1 part Virkon to 20 parts water)
JUMBO Liv-Wipes	Use according to the manufactures specifications.

Example - Use 5% Virkon (1 part Virkon to 20 parts of water)

- Use 25ml per 0.5 litres of water.
- Use 50ml per 1 litre of water.
- Use 100ml per 2 litres of water.
- Use 150ml per 3 litres of water.

Parait Road, Bell Block
PO Box 3003, Fitzroy
New Plymouth 4341
New Zealand
T. +64 (6) 755 0976
F. +64 (6) 755 0908

PO Box 2786
Taren Point
New South Wales 2229
Australia
T. 1800 120 727
F. 1800 120 717