

# Product Information

**TITLE:** Drugs of abuse test; urine dip Methadone (MTD) single panel  
**NPC:** HHH050  
**MPC:** HHSNGMTD  
**BRAND:** ONE STEP  
**SUPPLIER:** HOME HEALTH (UK) LTD

This test is for - Methadone (MTD).

## How to carry out a urine panel drug test


### DIRECTIONS FOR DRUG TEST USE

1. Pull the protective cap off the bottom of the test panel to reveal the absorbent testing strip, do not touch the strip and do not throw away the protective cap
2. Carefully dip the testing strip into the urine sample, ensuring that the urine does not come into contact with the plastic case of the test. Allow the test to absorb the urine for about 10-15 seconds.
3. Remove the test from the urine and replace the protective cap. Lay the test panel on a clean flat surface while the test line develops.
4. You can read a negative result as soon as it appears but a positive result must be read at least 5 minutes after removing from urine.
5. All results should be read within 10 minutes, otherwise a false reading may occur.



### INTERPRETATION OF RESULTS

You will see 2 letters on the panel by the test strip results window, 'C' this is the control band and 'T', this is the test band.

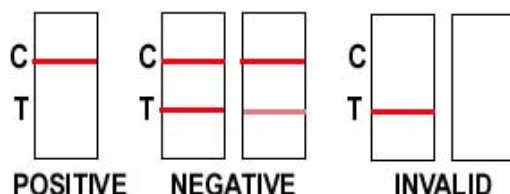

### (CONFIRM)

A pink/red coloured line should appear on each strip of the control band (C), this confirms each strip has worked properly. If no line appears in the control band (C) that strip hasn't worked properly and you may need to do a further test.

A second pink/red coloured line, no matter how faint should now appear in the test band (T),

this is a **negative** result. If no line appears in the test band (T) area, then the individual has tested **positive** for that drug of abuse.

The labels at the top of each test strip will tell you what drug each individual strip is testing for.

**Please remember that no matter how faint the Test line is, the test result is always classed as Negative.**